



DDP Holter Hook-Up Chart



LEGAL ALERT: Your Use of DDP equipment to Holter your dog constitutes your consent and agreement that the DDP may use and share your dog's Holter data for research purposes.

After Connecting your Dog:

1. Insert SD Flash Card into the Holter Monitor.
2. Insert one AA battery. Press ENTER when prompted.
3. Press ENTER to select NEW PATIENT.
4. Enter PATIENT ID (Enter your dog's microchip number. If not chipped, use the Embark code written on the small envelope with your dog's name). Use the green arrows to select a number, press ENTER for another digit.
5. Press EVENT when done.
6. Look at the ECG signals to check for a clean tracing for recording.
Also, check that the battery life is above 50%.
7. Press and hold EVENT for 3 seconds to begin recording.

After 24 Hours of Recording:

1. Remove battery from Holter.
2. Push chip IN to release it from the machine.
3. Place chip in small envelope with your dog's name on it.
4. Clean the Vest and dry thoroughly.
5. Return all materials (Holter, Pocket, Vest, and other Accessories) to the DDP as detailed in the DDP Holter Rental Contract.

